

ANOKRYO® Kombiset  
ANOKRYO® Gleitmittel  
ANOKRYO® Multi-Size

PZN 3941536  
PZN 7470157  
PZN 19075087

# ANOKRYO®

...cooling down  
the pain!



ANOKRYO  
still available  
as a combi set



**PakuMed®**  
medical products gmbh


Order directly at  
[www.pakumed.de](http://www.pakumed.de)

Im Löwental 79 · 45239 Essen | Germany  
Tel.: +49 201-43 70 97-0 · Fax: +49 201-43 70 97-29  
E-Mail: [info@pakumed.de](mailto:info@pakumed.de) · Internet: [www.pakumed.de](http://www.pakumed.de)

Rev. 2 (11.2023)

# ANOKRYO® Multi-Size

Basic help in treating proctologic disorders  
low temperature rod and dilatator

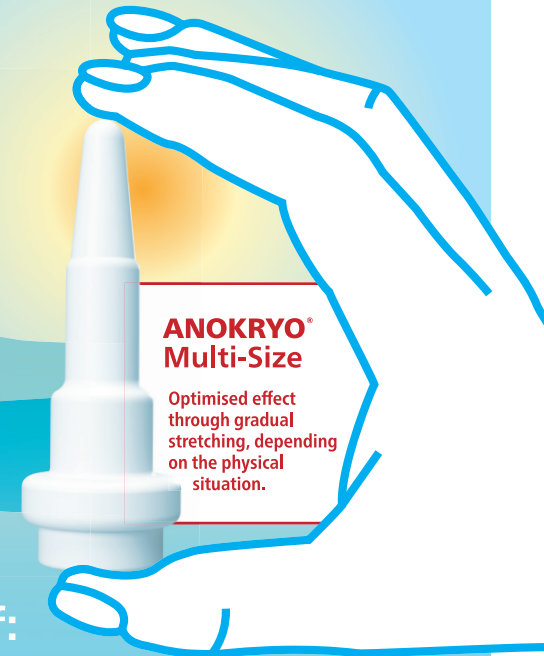

**ANOKRYO®  
Multi-Size**


Optimised effect  
through gradual  
stretching, depending  
on the physical  
situation.

For the treatment of:

- hemorrhoids 1° and 2°
- **ACUTE ANAL FISSURE**
- perianal thrombosis
- anal discomfort
  - post-operative pain
  - itching
  - anal stenosis
  - cramp
- in pregnancy and lactation
- at cryptitis, papillitis and after irradiation
- and associated constipation

## Usage instruction

The rod reaches its lowest temperature of minus (-15°C) after approx. 1 – 2 hours of storage in the freezer (-15° – -20°C). The rod is then lubricated with a water soluble lubricant and is subsequently inserted into the anal canal for 5 – 10 Min. After cleaning and disinfection the rod is once again stored in its case in the freezer. ANOKRYO® can be used repeatedly and contains no harmful contents, is odorless, hygienic and causes no staining.

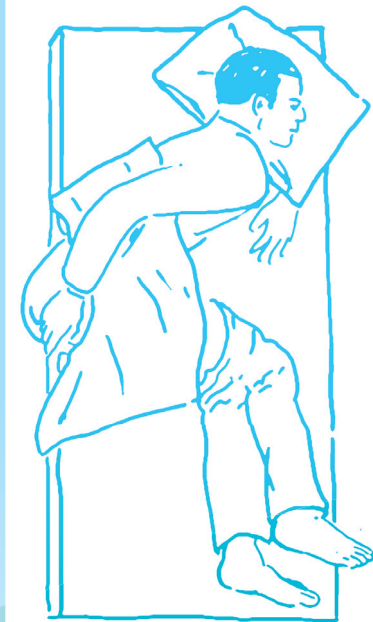

## Routine modes of treatment

hemorrhoid	anal fissure	perianal thrombosis and thrombosed hemorrhoidal prolapse
01. local preparations, ointment 02. sclerosing injections 03. rubber band ligation 04. infrared coagulation 05. laser treatment 06. cryotherapy 07. anal dilation 08. spincterotomy 09. hemorrhoidectomy 10. staplerhemorrhoidectomy 11. Lligation of the A. hemorrhoidalis	01. local preparations laxatives 02. infiltrations with local anesthetics, application of adstringents such as silver nitrate solution 03. quinine solutions 04. anal dilatation 05. sphincterotomy with or without excision of the fissure 06. application of nitroglycerine 07. botulin toxin	01. conservative treatment 02. operative treatment a) incision and thrombectomy b) excision c) hemorrhoidectomy d) dilatation

Those treating of these conditions are familiar with advantages and disadvantages of the above mentioned modes of therapy. Often a single procedure will not suffice to achieve healing. Most conditions do indeed have a tendency to heal spontaneously, this may however require a considerable amount of time, during with patients my have severe complaints.

Although the etiology of these conditions is not always clear, it appears that a permanent elevation of anal sphincter muscle tone is common to all, suggestion a contributing role in their development.

[www.anokryo.de](http://www.anokryo.de)



ANOKRYO® by combining low temperature with dilatation, offers an effective mode of treatment to safely and rapidly alleviate the patient's symptoms and accelerate healing.

This is achieved by cold application at temperatures of minus 15°C and dilatation to 15 mm with the ANOKRYO® rod. The duration of a single treatment is 5 – 10 minutes and should commence 2 – 3 times daily. Venous return is promoted and arterial inflow is reduced thus reducing sphincter muscle tone.

This is to turn relieves pain and inhibits inflammation.

**ANOKRYO® Multi-size allows for gradual introduction as needed.**

The use of low temperature application and dilatation in treating anal conditions is gaining popularity because it is effective, there are no untoward effects and is most suitable for home use.