Gebrauchsinformation

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ANOKRYO® Multi-Size

The low temperature rod and dilator for the treatment of hemorrhoids and their complications. ANOKRYO® is helpful for the following symptoms:

- pain relief
- inflammations
- bleeding
- itching
- accompanying anal fissures
- constipation
- anal sphincter

Patient information

Hemorrhoids are a disorder of the arterio-venous channels (arteries and veins) of the lower rectum. Approximately 70 % of all adults suffer from hemorrhoids during their lifetime. The majority of them cause various symptoms, e. g. pain and bleeding during defecation, thrombosis, eczema, itching, secretions, fungal infections, ulcerations, anal fissures and constipation.

Hemorrhoids can be treated effectively with ANOKRYO®, a low temperature rod with dilatation effect. ANOKRYO® is also suitable for the patient's own use in a domestic environment.

Instructions for use

1. The rod reaches its lowest temperature of – 15 °C after two hours of storage in the freezer. To ensure hygiene, the rod should always be stored in its case (without the lubricant gel).
2. Lubricate the rod evenly with gel especially at the tip to ensure smooth insertion. Please use only water-soluble lubricants as ANOKRYO® gel.
3. Carefully insert the rod while laying on the left side with the knees flexed and muscles relaxed. The rod should remain inserted for ten minutes.

The stretching effect of ANOKRYO® Multi-Size is improved by the graduated insertion depending on the anatomical situation. The rod must not be inserted further than the handle.

Within minutes, the temperature of the mucous membranes drops leading to immediate relief of pain, cessation of bleeding and reduction of swelling due to constriction of the blood vessels. Insertion can be painful when anal fissures are present and should be performed slowly and in a gentle manner. Relief of pain and healing is initiated by dilatation.

4. Cleaning of the rod with soap and water as well as routine disinfections is simple. After cleaning, the rod should once again be placed in the freezer.

5. Apply the ANOKRYO® rod at least twice a day. For severe pain, it can be used every hour or two hours without side effects. ANOKRYO® can be reused as often as required within the advertised shelf life. You may feel an unpleasant cold sensation within the first 30 seconds. This is normal and should not be a reason to remove the rod prematurely.

6. The ANOKRYO® rod contains no harmful contents, is odorless, hygienic, and causes no soiling.

Application fields / indications

- hemorrhoids (1st and 2nd degree)
- acute anal fissures
- perianal thrombosis
- postoperative pain in the anal area, in cryptitis / papillitis and after radiation treatments
- itching
- anal sphincter spasm with accompanying constipation
- anal symptoms during pregnancy and lactation

Patients with chronic symptoms must be carefully examined by a physician to exclude other serious disorders. Under certain circumstances the first use of ANOKRYO® by a doctor may be useful.

Contraindications / side effects

No untoward effects of ANOKRYO® application are known.

General recommendations for hemorrhoid diseases

- regulation of bowel movements by means of a high fiber diet
- meticulous cleaning and drying of the anal region after defecation
- light athletic activity (with the exception of bicycling and horseback-riding)
- avoidance of spicy food and alcohol

Ingredients / material

Rod: Polyethylene (PE-HD), latex-free

Gel: water, 7.5 % glycerol, 2 % methyl cellulose, 1.5 % benzyl alcohol

Note for users and patients: If you notice any damage to the rod or if coolant leaks from the rod, please dispose of the product with normal household waste. There is no danger to users or patients in the event of direct contact with the coolant.

Order information

ANOKRYO® Multi Size · PZN 19075087 (medical device)

ANOKRYO® lubricant gel · PZN 7470157 (medical device)

Note for users and patients: Please report any serious incident that has occurred in relation to this device to the manufacturer and to the competent authority of the Member State in which you are established.

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